

Fabric Face Covering -

This is a fabric face covering, it is designed to cover the nose and mouth and as a result can take the user some time to adjust to the sensation of wearing it. If you feel that your ability to breath normally is affected, please discontinue use.

The face covering is designed to be reused, please follow wash care instructions after each use.

This product is designed for adults only and should not be worn by children.

Please store in a cool, dry and well-ventilated place.

How to put on & fit the face covering -

For the best fit it is recommended that the face covering is worn on shaved skin.

Wash the face covering before use, please follow wash care instructions.

1. Wash your hands with soap and water or rub with a hand sanitiser before any handling of the face covering.
2. Select the appropriate size, either small or large.
3. Hold both ear loops, position the mask on the face and attach the loops around the ears.
4. Adjust the fit so the nose, mouth and chin are covered.
5. If you need to adjust the face covering once in place, please ensure you wash your hands with soap and water or rub with a hand sanitiser.

How to remove the face covering -

It is possible to contaminate your face or surroundings when removing the face covering. To avoid this, the recommendations are:

1. Wash your hands with soap and water or rub with a hand sanitiser.
2. Remove the face covering by holding the back of the elastic strips or fabric ties of the head harness without touching the front part of the face covering.
3. Place a face covering to be washed in a specific container (clean plastic bag).
4. Wash your hands with soap and water or rub with a hand sanitiser.

Wash and care instructions



40 Wash, Cool Iron, Do not Tumble, Do not use bleach, Do not dry clean, Do not use fabric softener

Washing with similar colours, reshape whilst damp, do not rub isolated stains.

Main – 92% Cotton 8% Elastane

WARNING - In order to use this face covering correctly, it is important that you follow these instructions.

WARNING - If you are ill, this face covering is unsuitable. Seek advice from your doctor.

WARNING - This face covering is not a medical device within the meaning of Directive 93/42/CEE or Regulation EU/2017/745 (surgical masks) nor is it personal protective equipment in the sense of Regulation EU/2016/425 (respiratory protective devices).

WARNING - Always check that the face covering is correctly fitted and covers your nose, mouth and chin. It is recommended that this face covering be worn on bare skin; beards can reduce the filtration efficiency to below the limits set out.

WARNING - This face covering does not replace protective measures (regular hand washing, physical separation, reduced contact with other people). It minimizes the projection of user's respiratory droplets saliva into the environment.

WARNING - Do not use when participating in vigorous physical activity.

WARNING - Stop using this product at the first signs of damage.

WARNING - Do not use dry cleaning and fabric softener.

WARNING - Clean the reusable face covering after each use.